CovClear Quick Reference Guide

For this test to work properly, read the instructions fully before starting the test and watch the instructional video (www.CovClearTest.com) For detailed instructions, refer to the instructions for use.

For the most up to date information on COVID-19, please visit www.canada.ca.

This home test kit has been authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens.

COLLECTION TIPS:

- Collect the sample as soon as possible after the onset of symptoms.
- Use only the swab provided.
- Follow the test procedure immediately after collecting the sample.
- Do not touch the tip (specimen collection area) of the swab.

POTENTIAL RISKS:



- Possible discomfort or other complication related to sample collection.
- Possible incorrect test result.

POTENTIAL BENEFITS:


- The result, along with other information, can help your healthcare provider make informed recommendations about your care.
- The results may help limit the spread of COVID-19 in your community.

HOW ACCURATE IS THE TEST?

- The CovClear COVID-19 Rapid Antigen Test was compared to a high-sensitivity FDA EUA RT-PCR test. The CovClear test correctly identified 99.4% of all results, including 96% (63 out of 66) positive results and 100% (411 out of 411) negative results.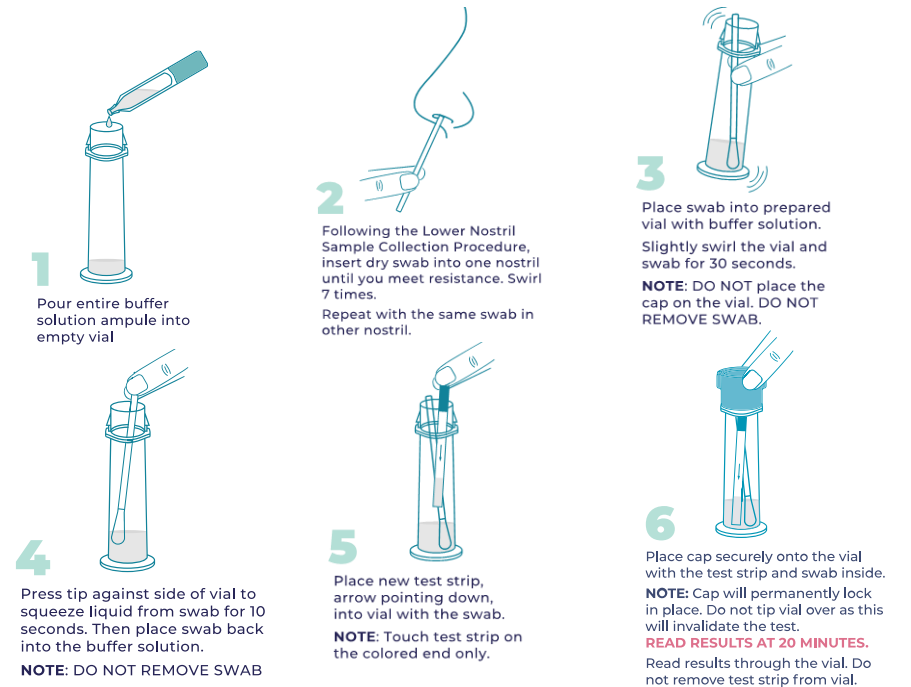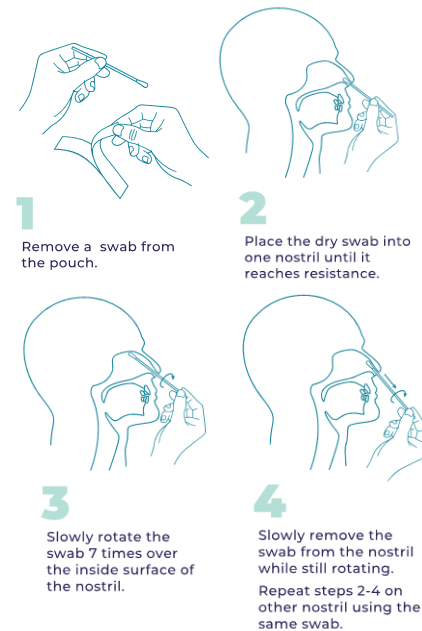

Results Key

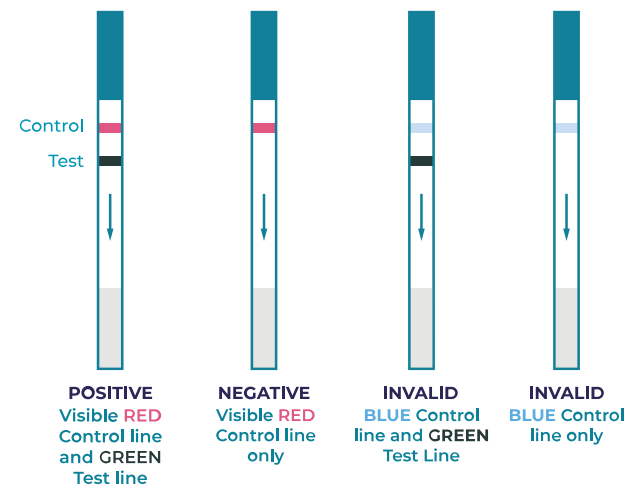Negative: A negative test result means that proteins have not been detected in your sample. It is possible for this test to give a negative result that is incorrect (false negative) in some people with COVID-19. This means you could possibly still have COVID-19 even though the test is negative. If you test negative and continue to experience COVID-19 symptoms of fever, cough and/or shortness of breath, seek follow-up care with your healthcare provider.

Positive: A positive result means that proteins from the virus that causes COVID-19 have been detected in your sample and it is very likely that you have COVID-19. You should self-isolate and seek follow-up care with your healthcare provider as additional testing may be necessary. There is a very small chance that this test can give you a positive test result that is wrong (false positive).

Invalid: An invalid result means that you must take a new test, including a new sample. If you collect a new sample, complete a new test, and receive a second invalid result, contact Empowered Diagnostics at +1 954-354-2768.

There is a higher chance of false negative results with home use tests than with laboratory-based molecular tests. This means that there is a higher chance this test will give you a negative result when you have COVID-19. Do not use this test as the only guide to manage your health. No matter what your test result is, please consult a healthcare professional if your symptoms persist or become more severe.

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